MASTER

14.04.2014

PRODUCT: NU-GEL®.

CODE: MNG415 & MNG415F.

PRODUCT DESCRIPTION: HYDROGEL WITH ALGINATE.

LANGUAGES: English, French, Italian, Spanish, Swedish, Dutch, Portuguese, Danish, Greek & Turkish.

SIZE: 348mm (H) x 256.5mm (W)
(Folded Size 87mm (H) x 64.125mm (W)).

COLOURS: PMS Process Blue & PMS Process Black.

DATE: 14.04.2014

AVC #: 0380285.i

BARCODE(S): None.

SYMBOLGY: N/A.

VERSION NUMBER: 06.




Leaflet prepared: March 2014

Do not use if package is damaged.
Do not reuse.
Do not sterilize.
The use by date of this product is printed on the packaging.

through the secondary dressing.
NU-GEL® Hydrogel should be changed if leakage occurs up to 3 days.
Depending upon the amount of wound exudate NU-GEL® Hydrogel may be left unsecured in place for up to 3 days.
Irrigate the wound with sterile saline to cleanse wound site.
• Dressing Change and Removal
• Any unused gel should be discarded.
• Cover with a suitable secondary dressing appropriate to the level of exudate.
• At least 5mm.
• Apply NU-GEL® Hydrogel to the whole wound to a depth of surface.
• Hold the concentra pack in one hand and direct NU-GEL® Hydrogel into the wound site by applying an even pressure to the base of the concentra pack with the thumb or heel of hand. The nozzle tip should be held clear of the wound surface.

EN Product Description
NU-GEL® Hydrogel is a transparent hydro active amorphous gel containing sodium alginate. The hydrogel creates a moist wound healing environment which assists with natural wound healing environments with medical components. NU-GEL® Hydrogel is indicated for the management and desloughing of wounds together with the management of chronic wounds throughout all stages of the healing process. If wound infection develops, appropriate antimicrobial therapy should be initiated. The use of NU-GEL® Hydrogel may be continued under medical supervision.
NU-GEL® Hydrogel contains propylene glycol which might cause irritation or sensitisation in a small number of patients. If such reactions are noted, use of NU-GEL® Hydrogel should be discontinued.

DIRECTIONS FOR USE
Application - see diagram, also on back of ampoule label
1. Carefully clean the wound.
2. Remove the label from the concentra pack by pulling the label as indicated.
3. Remove the cap, and swab the nozzle with a suitable antiseptic swab.
4. Twist to remove the tab.
NU-GEL® Hydrogel can be applied to the wound with a simple one-handed technique.

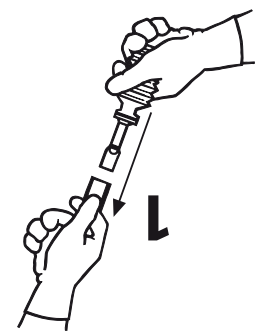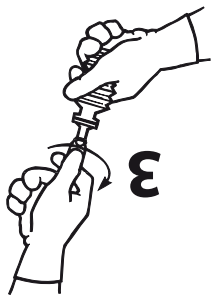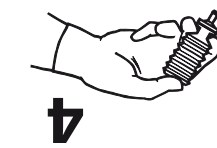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©2014 Systagenix Wound Management Limited, Gargrave, North Yorkshire, BD23 3RX, U.K.