PRODUCT: ADAPTIC® DIGIT.	SIZE: Sheet Size: 513mm (H) x 348mm (W) (Folded: 128.25mm (H) x 87mm (W)).
CODE: MAD003, MAD013, MAD023, MAD042 & MAD062.	COLOURS: PMS Process Blue & PMS Process Black.
PRODUCT DESCRIPTION: NON-ADHERING DIGIT DRESSING.	DATE: 07.06.2011
LANGUAGES: English, German, French, Italian, Spanish, Swedish, Dutch, Portuguese, Finnish, Danish, Greek, Norwegian, Polish, Hungarian, Czech, Slovakian, Slovenian, Latvian, Lithuanian, Estonian & Turkish.	AVC #: 5180345.g
	BARCODE(S): None.
	SYMBOLGY: N/A.
	VERSION NUMBER: 02.

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Leaflet prepared: May 2011

The use by date of this product is printed on the packaging.
Do not re-use.
Do not use if package is damaged.
Do not restitise.

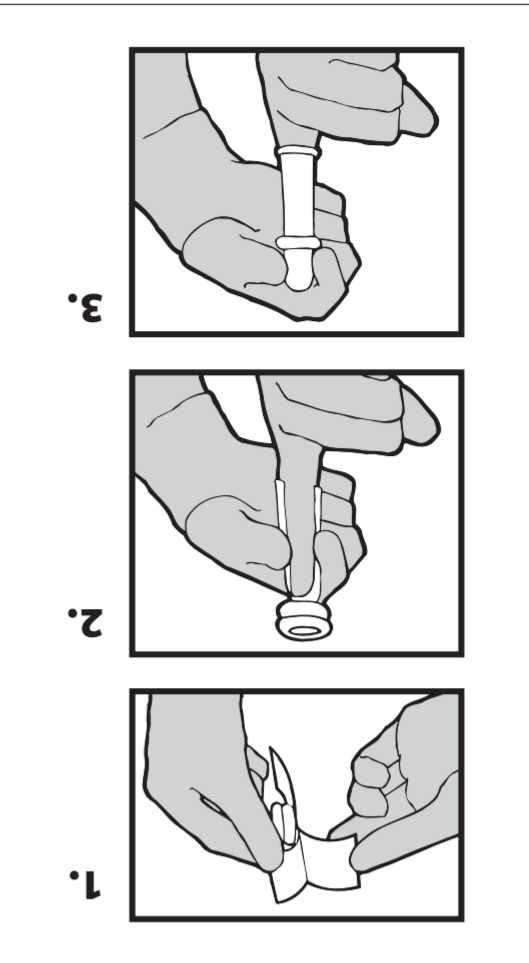
form a double layered tube for protection (see diagram 3).
secondary dressing if required). Follow this with the top ring to
3. Roll down lower ring of tube to expose primary strip and
smooth to cover any broken skin (see diagram 2).
2. Place primary non-adhering layer over digit tip and gently
1. Remove shiny backing strip (see diagram 1).

Directions for Use
1. Prepare the wound according to wound management
protocol.
2. Ensure skin surrounding the wound is dry.

Site Preparation
1. Prepare the wound according to wound management
protocol.
2. Ensure skin surrounding the wound is dry.

Indications
ADAPTIC® DIGIT dressing is indicated to manage a range of mild
to severe digit injuries.

EN Product Description
ADAPTIC® DIGIT Non-Adhering Digit Dressing is made of a unique
tacky silicone coated fabric with a bandage that is specifically
designed for dressing fingers and toes. The dressing does not
adhere to the wound, is conformable and non-bulky, therefore
allowing maximum digit flexibility. ADAPTIC® DIGIT is quick and
simple to apply and removes cleanly from the wound, minimising
trauma.

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©2011 Systagenix Wound Management Limited,