Pantone
Process
Blue

Pantone
Process
Black

MASTER

31.07.2013

PRODUCT: ADAPTIC®.

CODE: 2012, 2015 & 2019.

PRODUCT DESCRIPTION: NON-ADHERING DRESSING.

LANGUAGES: English, French, Italian, Spanish, Swedish, Dutch, Portuguese, Finnish, Danish, Greek, Norwegian & Turkish.

SIZE: (Sheet Size) 384.75mm (H) x 348mm (W)
(Folded: 128.25mm (H) x 87mm (W)).

COLOURS: PMS Process Blue & PMS Process Black.

DATE: 31.07.2013

AVC #: 552016S.a

BARCODE(S): None.

SYMBOLGY: N/A.

VERSION NUMBER: 05.

- (EN)** SYMBOLS USED ON LABELLING
(FR) SYMBOLES UTILISÉS SUR L'EMBALLAGE
(IT) SIMBOLI USATI SULLA CONFEZIONE
(ES) SIMBOLOS EMPLEADOS EN EL ENVASE
(SV) SYMBOLER ANVÄNDA PÅ FÖRPACKNINGEN
(NL) SYMBOLEN OP DE VERPAKKING
(PT) SÍMBOLOS UTILIZADOS NOS RÓTULOS
(FI) MERKITSEMISSÄ KÄYTETTÄVÄT SYMBOLIT
(DA) SYMBOLER ANVENDT PÅ EMBALLAGEN
(EL) ΣΥΜΒΟΛΑ ΧΡΗΣΙΜΟΠΟΙΟΥΜΕΝΑ ΣΤΗ ΣΥΣΚΕΥΑΣΙΑ
(NO) SYMBOLER SOM BRUKES PÅ MERKINGEN
(TR) ETİKETLERDE KULLANILAN SEMBOLLER

Do not use if package is damaged.
Ne pas utiliser si l'emballage est endommagé.
Non usare se la confezione è danneggiata.
No usar si el envase está dañado.
Används ej om produktförpackningen är skadad.
Niet gebruiken indien verpakking beschadigd is.
Não utilizar, se a embalagem estiver danificada.
Älä käyttää, jos pakkaus on vahingoittunut.
Må ikke anvendes hvis produktpakningen er beskadiget.
Na μη χρησιμοποιηθεί εάν η συσκευασία έχει καταστραφεί.
Må ikke brukes hvis emballasjen er skadet.
Ambalaj hasarlıysa kullanmayın.

Do not re-use (single use).
Ne pas réutiliser (usage unique).
Non riutilizzare (monouso).
No vuelva a utilizar (un solo uso).
Förbandet får inte återvändas (engångsbruk).
Niet opnieuw gebruiken (eenmalig gebruik).
Não reutilizar (utilização única).
Ei saa käyttää uudelleen (kertakäyttöinen).
Må ikke genbruges (engangsbrug).
Na μην επαναχρησιμοποιείται (μίας χρήσης).
Skal ikke brukes på nytt (engangsbruk).
Tekrar kullanmayın (tek kullanımlık).

Do not re-sterilize.
Ne pas restériliser.
Non risterilizzare.
No volver a esterilizar.
Får ej omsteriliseras!
Niet opnieuw steriliseren.
Não reesterilizar.
Ei saa steriloida uudelleen.
Må ikke reesteriliseres.
Na μην επαναποστειρώνεται.
Skal ikke reesteriliseres.
Yeniden sterilize etmeyin.

REF

Catalogue Number.
Numéro de catalogue.
Numero di catalogo.
Número de catálogo.
Katalognummer.
Catalogusnummer.
Código de produto.
Luettelonumero.
Katalognummer.
Αριθμός καταλόγου.
Katalognummer.
Katalog numarası.

STERILE R

Sterile. Method of sterilisation: irradiation.
Stérile. Méthode de stérilisation: irradiation.
Sterile. Metodo di sterilizzazione: raggi gamma.
Estéril. Método de esterilización: irradiación.
Sterilt. Steriliseringmetod: strålning.
Steriel. Sterilisatiemethode: bestraling.
Esterilizado. Método de esterilização: radiação.
Steriili. Sterilointimenetelmä: säteilysterilointi.
Steril. Steriliseringmetode: Bestråling.
Αποστειρωμένο. Μέθοδος αποστείρωσης: ακτινοβολία.
Steril. Steriliseringmetode: R.
Steril. Sterilizasyon yöntemi: R (Radyasyon).

Consult instructions for use.
Consulter le mode d'emploi.
Consultare le istruzioni per l'uso.
Consulte las instrucciones de uso.
Läs instruktionerna innan produkten tas i bruk.
Raadpleeg de gebruiksaanwijzing.
Consultar as instruções de utilização.
Katsso käyttöohjeita.
Læs brugsvejledningen.
Συμβουλευθείτε τις οδηγίες χρήσης.
Se bruksanvisningen.
Kullanım için talimatlara bakın.

CE 0086

CE-mark and identification number of notified body.
Marquage CE et n° identification de l'organisme notifié.
Marchio CE e numero di identificazione dell'Organismo notificato.
Marca CE y Número de identificación del organismo notificado.
CE-märke och identifieringsnummer för Anmält Organ.
CE-markering en identificatienummer van de Aangemelde Instantie.
CE-markering en identificatienummer van de Aangemelde Instantie.
Marca CE e número de identificação do organismo notificado.
CE-merkintä ja ilmoitetun laitoksen tunnusnumero.
CE-mærkning og identifikationsnummer for Bemyndiget Organ.
Ενδειξη CE και αριθμός φορέα που έχει ενημερωθεί.
CE-merke og identifikationsnummer på varslet organ.
CE işareti ve yetkili kurumun kod numarası.

LOT

Batch number.
Numéro du lot.
Numero di Lotto.
Número de lote.
Satsnummer.
Lotnummer (partij).
Número de lote.
Eränumero.
Batchnummer.
Αριθμός παρτίδας.
Batchnummer.
Parti no.
Use by: year and month.
A utiliser avant: année et mois.
Da usarsi entro: anno e meso.
Fecha de caducidad: año y mes.
Användes senast: år och månad.
Uiterste gebruiksdatum: jaar en maand.
Validade: ano e mês.
Käytettävä viimeistään: vuosi ja kuukausi.
Anvendes senest: år og måned.
Χρήση μέχρι: έτος και μήνας.
Brukes innen: år og måned.
Son kullanma tarihi: yıl ve ay.

Store below 40°C/104°F.
Conservar en dessous de 40°C/104°F.
Conservare a T < 40°C/104°F.
Almacenar a temperatura inferior a 40°C/104°F.
Förvaras under 40°C/104°F.
Bewaren onder 40°C/104°F.
Armazenar abaixo de 40°C/104°F.
Säilytettävä alle 40°C/104°F.
Opbevares under 40°C/104°F.
Αποθήκευση κάτω από 40°C/104°F.
Lagres under 40°C/104°F.
40°C/104°F'nin altında saklayın.

Manufacturer.
Fabricant.
Fabbricante.
Fabricante.
Tillverkare.
Producent.
Valmistaja.
Fabrikant.
Κατασκευαστής.
Produsent.
Üretici.

See instructions for use for exudate level.
Pour plus d'informations sur les niveaux d'exsudats, lire le mode d'emploi.
Per il livello di essudato, vedere le istruzioni per l'uso.
Consulte las instrucciones de uso sobre el nivel de exudado.
Se även instruktionerna om användning till exuderande sår.
Zie gebruiksinstructies voor exsudaatniveau.
Consultar as instruções de utilização para o nível de exsudado.
Katsota haavanestein määrää käyttöohjeista.
Se anvisningerne for brug til eksudatniveau.
Βλ. τις οδηγίες χρήσης για το βαθμό εξιδρωσης.
Se bruksanvisningen for utsodringsnivå.
Akıntı düzeyi için kullanım talimatlarına bakın.

552016S.a

©2013 Systagenix Wound Management Limited,
Gargrave, North Yorkshire, BD23 3RX, U.K.

CE
0086

Let's Comfort®

ADAPTIC®

NON-ADHERING DRESSING

PANSEMENT NON-ADHERENT
MEDICAZIONE NON-ADERENTE
APÓSITO NO ADHERENTE
ICKE VIDHÄFTANDE FÖRBAND
NIET VERKLEVENDE KOMPRES
COMPRESSA NÃO-ADERENTE
TARTTUMATON TAITOS
IKKE-HÆFTENDE FORBINDING
MH AYTKOKΛΗΤΟ ΕΠΙΘΕΜΑ
IKKE-KLEBENDE BANDASJE
YAPIŞMAZ PANSUMAN

STERILE R

Systagenix

Pantone
Process
Blue

Pantone
Process
Black

MASTER

31.07.2013

EN Product Description

ADAPTIC® Non-Adhering Dressing is a primary dressing made of knitted cellulose acetate fabric and impregnated with a specially formulated petrolatum emulsion. It is designed to help protect the wound while preventing the dressing from adhering to the wound.

- Helps prevent dressing adherence
- Helps protect regenerating tissue and minimizes patient pain and trauma during dressing changes
- Helps prevent the pooling of fluid at the wound site
- Exudate easily passes through to the secondary absorbent dressing
- Can be cut to wound size without unravelling and shredding

Indications

ADAPTIC® Non-Adhering Dressing is indicated for dry to highly exuding wounds where adherence of dressing and exudate is to be prevented including:

- 1st & 2nd degree burns
- Abrasions
- Grafts
- Venous ulcers
- Pressure ulcers
- Nail extractions
- Eczema
- Staples
- Surgical incisions
- Lacerations
- Reconstructive procedures
- Suture lines

DIRECTIONS FOR USE

Site Preparation

1. Prepare the wound according to wound management protocol.
2. Ensure skin surrounding the wound is dry.

Dressing Application

1. Carefully clean the wound following facility protocol.
2. Cut ADAPTIC® Dressing to size if required and place directly on the wound.
3. Cover with sterile secondary dressing of your choice depending on the amount of wound exudate.
4. ADAPTIC® Dressing may be left in place for several days depending upon amount of exudate.

Do not use if package is damaged.

Do not reuse.

Do not resterilize.

The use by date of this product is printed on the packaging.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a properly licensed healthcare practitioner.

This caution is not applicable outside the U.S.

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PRODUCT: ADAPTIC®.

CODE: 2012, 2015 & 2019.

PRODUCT DESCRIPTION: NON-ADHERING DRESSING.

LANGUAGES: English, French, Italian, Spanish, Swedish, Dutch, Portuguese, Finnish, Danish, Greek, Norwegian & Turkish.

SIZE: (Sheet Size) 384.75mm (H) x 348mm (W) (Folded: 128.25mm (H) x 87mm (W)).

COLOURS: PMS Process Blue & PMS Process Black.

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