MASTER

10.06.2016

PRODUCT: BIOCLUSIVE™ PLUS.

CODE: BIP0607, BIP1012, BIP1520 & BIP2030.

PRODUCT DESCRIPTION: TRANSPARENT FILM DRESSING.

LANGUAGES: English, German, French, Italian, Spanish, Dutch, Portuguese, Danish, Polish, Czech & Turkish.

SIZE: Sheet Size: 440mm (H) x 340mm (W).
Folded Size: 110mm (H) x 85mm (W).

COLOURS: PMS 2695 (Purple) & PMS Cool Gray 11.

DATE: 10.06.2016

AVC #: WMF0125(P)-3

BARCODE(S): None.

SYMBOLGY: N/A.

VERSION NUMBER: 03.

EN) PRODUCT DESCRIPTION
 BIOCLUSIVE™ PLUS transparent film dressing is a self-adhesive dressing consisting of a transparent semi-permeable polyurethane film coated with a tacky adhesive.
 BIOCLUSIVE™ PLUS provides a bacterial barrier which also allows the transmission of moisture vapour and oxygen. BIOCLUSIVE™ PLUS allows visual inspection of the wound area and is flexible and conformable for use on difficult contours.
INDICATIONS
 BIOCLUSIVE™ PLUS dressing is indicated for the management of wounds where there is no exudate, or light levels of exudate.
 BIOCLUSIVE™ PLUS dressing should be used under health care professional direction for the following indications:
 • Minor burns
 • Donor sites
 • Superficial pressure areas and leg ulcers
 • Cuts and abrasions
 • Scalds of shallow depth
 • Reducing shear and friction
 • Reducing the risk of infection
 BIOCLUSIVE™ PLUS is also suitable for:
 • Protection of patients with fragile skin. Care should be taken not to dislodge catheters or other devices.
 • Dressing change and removal
 BIOCLUSIVE™ PLUS dressing may be left in place for several days depending upon amount of exudate.
 - Dressing change requires it to be changed by the patient or caregiver.
 - Do not use if the dressing is damaged.
 Do not use if package is damaged.
 Do not re-sterilize.
 The use of this product is printed on the packaging.
 Caution: Federal Law (USA) restricts the device to sale by or on the order of a properly licensed healthcare practitioner. This caution is not applicable outside the U.S.
PRECAUTIONS
 If initial infection is observed during the use of BIOCLUSIVE™ PLUS dressing, discontinue use and refer to local protocol for managing the infection.
DIRECTIONS FOR USE
 adhesion to skin will also compromise the bacterial barrier.
 2. Ensure that the skin surrounding the wound is clean, dry and free from grease, soaps or detergents.
 1. Prepare the wound according to local wound management protocol.

DESSING APPLICATION:
 1. Choose an appropriate size and remove the dressing.
 2. Peel open the adhesive and remove the dressing.
 3. Remove the large, central release from the adhesive underside of the dressing (marked A).
 4. Position the dressing centrally over the wound, above the adhesive underside of the dressing (marked A).
 5. Press the exposed adhesive side of the dressing onto the skin surrounding the wound.
 6. Peel away the top release liner tabs from the underside of the dressing (marked B).
 7. Press away the top and press the remaining edges of the dressing onto the skin using a rolling motion.
 8. Ensure the dressing covers the entire wound.
 9. Once the dressing is secure, grasp the upper release liner tabs (marked C) and peel away from the centre of the dressing openings.
DESSING CHANGE AND REMOVAL
 BIOCLUSIVE™ PLUS dressing may be left in place for several days depending upon amount of exudate.
 - Dressing change requires it to be changed by the patient or caregiver.
 - Do not use if the dressing is damaged.
 Do not re-sterilize.
 The use of this product is printed on the packaging.
 Caution: Federal Law (USA) restricts the device to sale by or on the order of a properly licensed healthcare practitioner. This caution is not applicable outside the U.S.
PRECAUTIONS
 If initial infection is observed during the use of BIOCLUSIVE™ PLUS dressing, discontinue use and refer to local protocol for managing the infection.
DIRECTIONS FOR USE
 adhesion to skin will also compromise the bacterial barrier.
 2. Ensure that the skin surrounding the wound is clean, dry and free from grease, soaps or detergents.
 1. Prepare the wound according to local wound management protocol.

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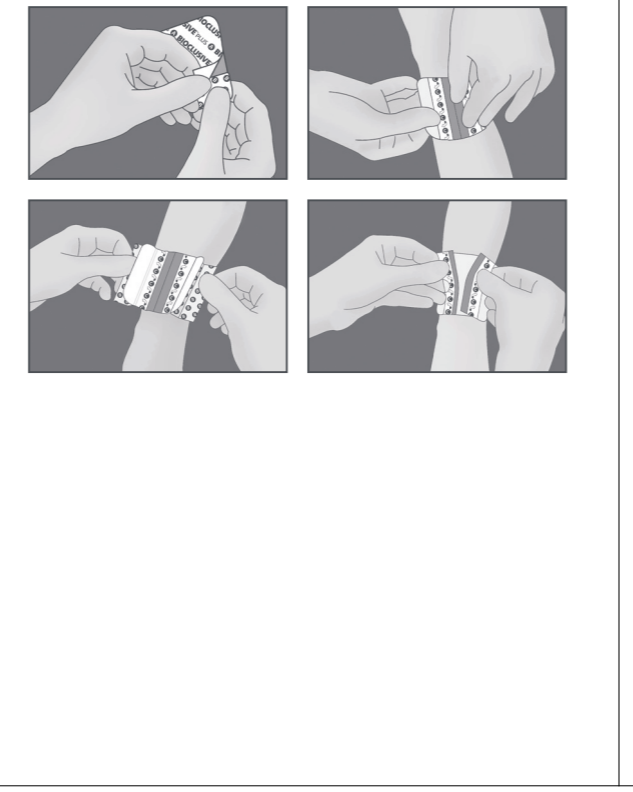
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
Acelity™
 WMF0125(P)-3
 Distributed by:
 ©2016 Systagenix Wound Management Limited,
 Gargrave, North Yorkshire, BD23 3RX, UK
 Made in the USA
 Systagenix
 An Acelity Company
 1016-05