## BLOOD GLUCOSE & KETONES MONITORING SYSTEM

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### IMPORTANT SAFETY INFORMATION READ BEFORE USE

- Use the meter and separate components ONLY for the intended purpose as described in the enclosed user guides.
- DO NOT use any accessories and/or components not described in the enclosed user guides.
- Do not use the meter or separate components if damaged and/or not working properly.
- This meter is NOT a medical treatment device. The measured data should only be interpreted as a reference. Always consult your doctor and/or diabetes nurse for accurate interpretation of the measured values.
- Before using the meter, read all accompanying instructions and carefully practise the procedure. Perform all quality checks as described.
- Keep the meter and separate components out of reach of young children. If swallowed, small components could create a choking hazard.
- If using the meter in a very dry environment and particularly when synthetic materials are present (clothing, carpet, etc.), take care to avoid a static discharge, especially when touching the meter and test strips. If the meter and/or test strips are exposed to a static discharge they may produce incorrect measurement results.
- Do not use the meter in the vicinity of strong electromagnetic radiation. This could have an adverse impact on accuracy.
- Good maintenance and regular tests with the control fluid are important for the reliability and life of your meter. If you have concerns about the accuracy of your measurements, contact customer service or your supplier.

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