
Job No: 13519
File Name: KFGB IFU
Date Created: 26.6.13
Client: Crawford Healthcare
Client Contact: Caroline Green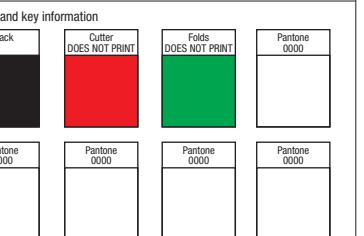

Cutter details:
Cutter file size: 300mm deep x 600mm wide FLAT
Finished size: 150mm deep x 100mm wide FOLDED

Supplied by:

Key information:

Packaging type:

Description:

Substrate:

Other information:

	Initials	Approved	To Amend
Operator	TE		
Proof Reader			
Typography			
Final Proof			

Comments:

The Foundry Communications Ltd 3 Denmark St Goose Green Altrincham Cheshire WA14 2SS T+44 (0)161 926 8444 F+44 (0)161 926 8448 E mail@foundrycomms.co.uk WWW.foundrycomms.co.uk


KerraFoam™ Gentle Border

Super-absorbent foam dressing

SILICONE ADHESIVE


Instructions for use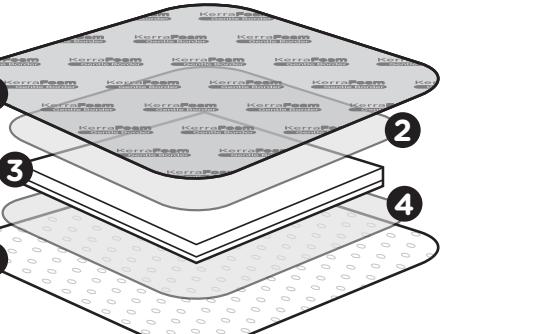


Super-absorbent foam dressing

- Outer polyurethane film layer is water resistant and provides a bacterial barrier
- Soft, nonwoven material
- Super-absorbent core and foam pad absorb and lock in exudate
- Unique horizontal wicking layer efficiently distributes exudate, reducing strike-through
- Patterned, gentle, silicone adhesive wound contact layer

How to apply KerraFoam Gentle Border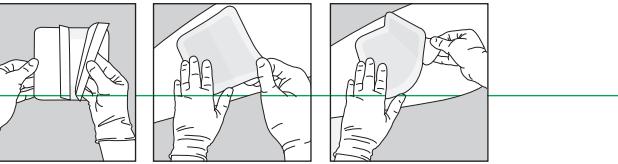

How to apply KerraFoam Gentle Border - Heel

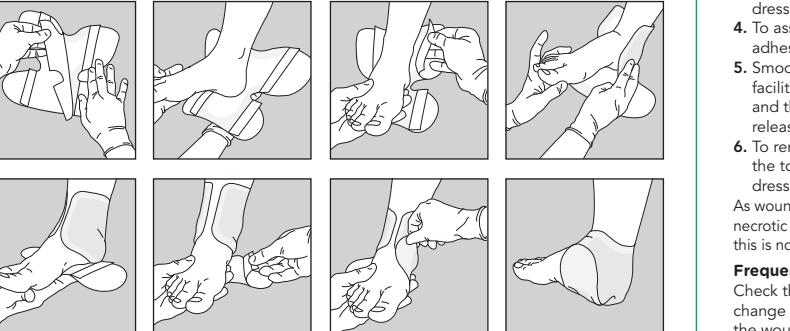How to apply KerraFoam Gentle Border - Sacrum

Product Description

KerraFoam™ Gentle Border is a sterile, super-absorbent foam dressing. KerraFoam Gentle Border absorbs and retains exudate (locks in), maintains a moist wound environment and helps reduce the risk of skin maceration. The absorption capacity of KerraFoam Gentle Border allows fewer dressing changes, making it cost-effective as well as causing fewer disturbances to the wound. Fewer dressing changes and the soft silicone adhesive ensure less pain at dressing change for the patient.

The dressing is secured to the wound through the soft silicone adhesive, a secondary dressing is not required. This layer adheres to dry areas of the skin but not moist areas such as the open wound. The dressing is very conformable and adheres to awkward areas of the body comfortably. The adhesive reduces leakage and helps stop exudate spreading to healthy skin. The top polyurethane layer also assists in creating a moist environment essential for wound healing to occur.

For single use only.

Indications

Suitable for use on moderate to heavily exuding wounds such as:

- Leg ulcers
- Pressure ulcers
- Diabetic foot ulcers

Also suitable for other similar wounds producing moderate to high levels of exudate.

The dressing will also absorb exudate under compression.

Contraindications

Known allergy or hypersensitivity to any of the components of KerraFoam Gentle Border:

- Sodium polyacrylate
- Viscose
- Polyethylene terephthalate
- Vinyl acetate
- Polyurethane
- Silicone

KerraFoam Gentle Border should not be used on:

- Eyes, mucous membranes or tendons, due to the risk of dehydration
- Any cavity or sinuses, as the dressing will swell on absorption
- Dry wounds, as the absorption capacity may cause sticking to the wound or cause the wound to dry out and become painful

KerraFoam Gentle Border is not a haemostat and should not be used on heavily bleeding wounds.

Precautions/Warnings

- For external use only
- Do not use if dressing is damaged
- Do not re-use. Re-use will cause cross-contamination
- Do not cut the dressing – the dressing must remain intact
- In case of clinical infection, consult a healthcare professional
- Dispose in normal waste if used in a household
- Dispose in special clinical waste if used in a clinical establishment
- The use of oil-based products (e.g. lotions/ointments) may reduce the ability of the dressing to adhere to the skin and may reduce the absorption capacity of the dressing

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Using KerraFoam Gentle Border

1. Cleanse the wound in accordance with normal procedures. The peri-wound skin should be clean and dry to assist adhesion.

2. Select an appropriate dressing size, so the wound dressing pad overlaps the wound margin by approximately 2cm/0.8".

3. Start to remove one half of the release liner on the adhesive covering and apply the dressing to the wound.

4. To assist adhesion, gently press the adhesive to the skin.

5. Smooth the dressing over the wound to facilitate a good contact between the pad and the wound, and remove the remaining release liner.

Frequency of Change

Check the dressing at regular intervals and change according to the clinical condition of the wound, when saturated or if the film has lifted/peeled off. If the condition of the wound deteriorates, discontinue use and consult your healthcare professional.

KerraFoam Gentle Border can be left in place for several days. However, this depends upon the condition of the wound and the skin in the peri-wound area.

Notes

KerraFoam Gentle Border can be ordered from authorised medical distributors and from pharmacies.

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