
KerraLite Cool™

Soothing, debriding and moisture-balancing dressing

NON-ADHESIVE

Instructions for use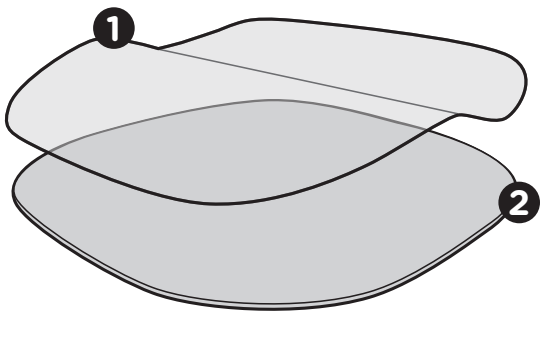

1. Fluid-repellent, flexible and highly breathable polyurethane film layer
2. Pro-ionic™ gel contact layer soothes, debrides and maintains a moist healing environment

Product Description
KerraLite Cool has been formulated and designed specifically to encourage wound bed preparation and granulation of chronic wounds, while minimising pain levels. The dressing helps to provide an environment for optimal wound healing by managing wound exudate levels and protecting against wound dehydration and external bacterial contamination. The gel provides both cushioning and absorption. The partially-hydrated nature of the dressing aids the process of autolytic debridement and helps soothe the wound on contact.

Indications
Under the supervision of a healthcare professional, KerraLite Cool may be used for the management of chronic wounds, on painful wounds and skin conditions such as leg ulcers, radiation therapy damage, burns and scalds. It may be used on low-exuding and non-exuding wounds to assist in autolytic debridement by hydration of necrotic and sloughy tissue and for absorption of exudate. It may be used on moderately exuding wounds, although in such cases it may be preferable to cut the dressing to fit within the wound margins. The dressing can be used on contaminated and colonised wounds. It may be used with secondary dressings where appropriate. KerraLite Cool can be used under compression.

Contraindications
KerraLite Cool is not indicated for full-thickness wounds, heavily bleeding wounds, third degree burns, or as a covering for deep, narrow cavities or sinuses.

Precautions/Warnings
• Clinical history, requisites anatomical and should be assessed by a Clinician
• In the case of an intolerance to the dressing, remove and clean the area carefully
• During the body's natural healing process, non-viable tissue is removed from the wound (autolytic debridement), which could initially make the wound appear larger. If the wound continues to grow larger after the first few dressing changes, consult a healthcare professional. The wound should be inspected during dressing changes. Consult a healthcare professional if there are signs of: infection (increased pain, increased redness, increased wound fluid levels); excessive bleeding; an unexpected change in wound colour and/or odour; irritation (increased redness and/or inflammation); no signs of healing
• Do not use on patients with a known sensitivity to hydrogel (allergic reaction)
• Do not re-use in whole or in part, as the sterility/performance of the dressing may be compromised
• The gel layer in KerraLite Cool is absorbent, and can tend to break up on prolonged exposure to exudate. This is normal

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Kerraford Healthcare™ is a trade mark of Crawford Healthcare Ltd
© Copyright Crawford Healthcare Ltd, 2018.
Pro-ionic™ is a trade mark of First Water Limited.