
KerraMax care™

Super-absorbent dressing

NON-ADHESIVE

Information for use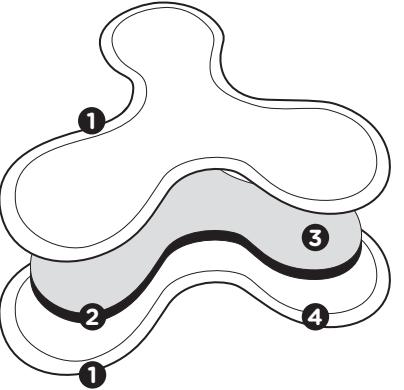


Super-absorbent dressing


1. Soft, nonwoven material for comfort
2. Super-absorbent core locks in exudate
3. Unique horizontal wicking layer
4. Heat-sealed border to prevent exudate leakage

How to apply KerraMax™ Care Multisite:


Breast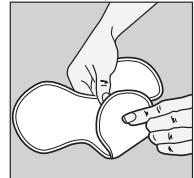


Shoulder

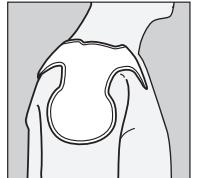
Groin

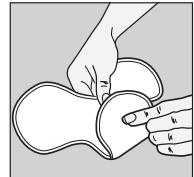
Elbow

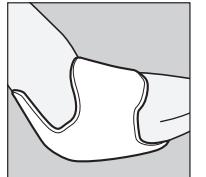Heel

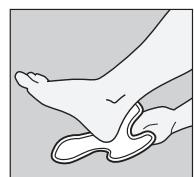Hip

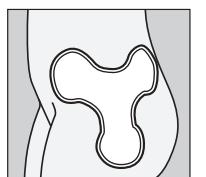Product Description

KerraMax Care™ Multisite is a sterile, super-absorbent dressing for use on exuding chronic wounds that are difficult to dress.

The shape of KerraMax Care Multisite enables the dressing to be positioned on areas of the body such as e.g. heel, groin, elbow, armpit (axilla), hip, shoulder, chest, breast, behind the knee.

Excess chronic wound fluid can delay the healing process. KerraMax Care Multisite absorbs exudate and locks it away inside the dressing, helping to reduce skin maceration.

High absorption and retention enable fewer dressing changes and minimise disturbance to the wound.

Indications

Suitable for use on exuding wounds, particularly those wounds that are difficult to dress e.g. heel, groin, elbow, armpit (axilla), hip, shoulder, chest, breast, behind the knee.

KerraMax Care Multisite can be used as a primary or secondary dressing.

KerraMax Care Multisite is suitable for use under all forms of compression.

Contraindications

Known allergy or hypersensitivity to any of the components of KerraMax Care Multisite. There are no known side effects of KerraMax Care Multisite.

- Sodium polycrylate
- Viscose
- Polyethylene terephthalate
- Vinyl acetate

KerraMax Care Multisite should not be used on:

- Eyes, mucous membranes or tendons, due to the risk of dehydration
- Any cavity or sinuses, as the dressing will swell on absorption
- Dry wounds, as the absorption capacity may cause sticking to the wound or cause the wound to dry out and become painful

KerraMax Care Multisite is not a haemostat and should not be used on bleeding wounds.

Precautions/Warnings

- For single use only
- For external use only
- Do not re-use. Re-using will cause cross contamination
- Do not cut the dressing
- Dispose in normal waste if used in a household
- Dispose in special clinical waste if used in a clinical establishment

Using KerraMax Care Multisite

1. Clean the wound according to local procedures.

2. The dressing pad needs to overlap the edge of the wound.

3. Open the pack and lay the sterile pad onto the wound. Either side of the sterile pad may be used.

4. Lightly cover with a bandage, retention dressing or tape to secure the dressing.

Frequency of change

Check the dressing at regular intervals and change according to the clinical condition of the wound, or when saturated. If the condition of the wound deteriorates, discontinue use and consult your healthcare professional. KerraMax Care Multisite can be left in place for several days. However, this depends upon the condition of the wound and the skin in the peri-wound area.

Notes

KerraMax Care Multisite can be ordered from authorised medical distributors and from pharmacies.

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