## 3M™ Tegaderm™ Foam Adhesive High Performance Foam Adhesive Dressing

**Description:**  
3M™ Tegaderm™ High Performance Foam Adhesive Dressing is a highly absorbent, breathable wound dressing. It is constructed from a conformable polyurethane foam pad, additional absorbent nonwoven layers, and a top layer of transparent adhesive film. This dressing is a moisture and vapor permeable wound dressing, with a top layer of transparent adhesive film.

It is designed to reduce the chance of bacterial strike-through and acts as a barrier to outside contamination, including bacteria and viruses. The dressing maintains a moist wound environment, which has been shown to enhance wound healing; the dressing is sterile.

### Indications for Use:

Tegaderm™ Foam Adhesive dressing is designed to allow easy, one-handed application to difficult body contours, including heels and elbows, without the need for cutting. The dressing is indicated for use as a primary dressing for low to highly exuding, partial and full thickness dermal wounds, including:

- pressure ulcers
- venous leg ulcers
- abrasions
- superficial partial-thickness burns
- donor sites
- cellulitis
- skin tears
- neuropathic ulcers

### Additional applications of Tegaderm™ Foam Adhesive dressing include:

- use as a secondary (cover) dressing in conjunction with wound fillers (such as gauze or alginate dressings).
- use under compression wrap systems for venous leg ulcer treatment.

This product is not designed, sold or intended for use except as indicated. Tegaderm™ Foam Adhesive dressing is not intended for use in pressure reduction or as a surgical sponge.

### Contraindications:

None known

### Warnings:

None known

### Precautions:

1. Do not use Tegaderm™ Foam Adhesive dressing in combination with oxidizing agents such as hydrogen peroxide or hypochlorite since they may affect the structure and performance of the polyurethane foam.

2. At the time of dressing change, if the dressing is adhered to the wound surface, saturate with normal saline and gently loosen.

3. Observe the signs of infection. If infection occurs, signs which may be fever, increased pain, redness, swelling or an unusual odor or discharge, see a health care professional. Tegaderm™ Foam Adhesive dressing may be used on infected wounds only if the wound does not begin to show signs of infection or if any other unexpected symptoms occur, consult a health care professional.

### Directions for Use:


Note: Follow facility guidelines for infection control.

#### Before the Dressing:

1. Clip excess hair for patient comfort.
2. Cleanse wound and surrounding skin according to facility policy. Note: If perlwend skin is fragile or exposure to wound exudate is likely, apply a barrier film to the wound film to dry before dressing application.
3. Assess wound: Select appropriate size dressing to ensure that the foam pad is larger than the wound area.

#### Applying the Dressing:

1. Hold the dressing by the side tabs and remove the printed liner, exposing the adhesive border (Figure 1).
2. Hold the dressing by one of the side tabs and remove the printed liner, exposing the adhesive border (Figure 2).
3. While holding the side tab, position the dressing over the wound (Figure 3).
4. Gently press the adhesive film border to the skin (Figure 2). Avoid stretching the dressing or the skin.
5. Beginning at the center of the dressing, grasp one section of the paper carrier and pull toward the outer edge of the dressing. Repeat this for each of the remaining carrier sections until all paper carrier is removed (Figure 3).
6. Firmly press and conform the adhesive border to the skin (Figure 4).

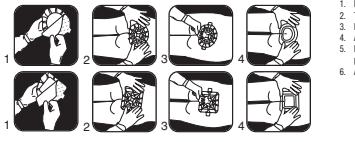#### Heel/Elbow Application

1. Remove dressing from package.
2. Hold the dressing by one of the side tabs and remove the printed liner, exposing the adhesive border (Figure 1).
3. While holding the side tab, position the dressing over the wound (Figure 2).
4. With one hand, "cup" the dressing onto the skin. Continue to press and conform the dressing and adhesive border onto the skin (Figure 3).
5. Beginning at the center of the dressing, grasp one section of the paper carrier and pull toward the outer edge of the dressing. Repeat this for each of the remaining carrier sections until all paper carrier is removed (Figure 4).
6. Firmly press and conform the adhesive border to the skin (Figure 5).

#### Removing the Dressing:

1. Carefully lift the dressing edges from the skin. It is difficulty lifting the dressing, apply tape to the edge of the dressing and the tape to it.
2. Continue lifting the edges until all are free from the skin surface. Remove the dressing.

#### Dressing Change Considerations:

1. Observe the signs of infection. As the dressing absorbs, exudate willwick to the top of the dressing, and exudation may be noted. When the exudate spreads to the edges of the dressing or the dressing begins to leak, a dressing change is indicated.
2. Dressing change frequency will depend on the type of wound, volume of exudate, and clinical situation. Change at least every seven days, or as indicated by treatment protocol.

#### Storage and Shelf Life:

For best results, store in a cool, dry place at room temperature. Avoid excessive heat and humidity. For shelf life, refer to the expiration date printed on each box.

#### How Supplied:

Supplied in boxes of individually packaged sterile dressings. Sterility of product is guaranteed unless the individual package is damaged or open.

If you have any questions or comments, in the USA please contact the 3M Health Care Customer Helpline at 1-800-228-3957.

In Canada, contact 3M Canada Company, P.O. Box 5757, London, Ontario, N6A 4T1, 1-800-364-3577. Outside the United States, contact your local 3M subsidiary.

\*In vitro testing shows that the transparent film provides a viral barrier from viruses 27 nm in diameter or larger while the dressing remains intact without leakage.

#### Explanation of Symbols:

- (S) Do not use if package is damaged or open  
(R) This product and package do not contain natural rubber latex.  
(A) Caution, see instructions for use  
(D) Do not reuse  
(U) Use by date  
(LOT) Batch code  
(M) Manufacturer  
STERILE | R Sterilized using irradiation

#### CE 0086

Made in U.S.A. by  
3M Health Care  
2510 Conway Ave.  
St. Paul, MN 55144  
1-800-228-3957  
www.3m.com/healthcare

3M and Tegaderm are trademarks of 3M.

© 2012, 3M. All rights reserved.

Because the differences in scanning equipment, SGS Minnesota cannot guarantee the scanability of UPC bar codes. The Uniform Code Council recommends a UPC symbol be printed black or white at 100% with no truncation. All UPC codes are test scanned and verified before leaving our facility.

ECIREP 3M Deutschland GmbH  
Health Care Business  
Carl-Schurz-Str. 1  
41453 Neuss, Germany

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Issue Date 2010-03

34-8714-2922-0

**Southern Graphic Systems**  
9300 WINNETKA AVE N, BROOKLYN PARK, MN 55445, 763.488.7000

Proof prepared for 3M Company REV 01 Created in Adobe InDesign CS5

DESC: 3M™ Tegaderm™ - High Performance Foam Adhesive Dressing - Insert

DATE: CHANGES  
03.10.14 Update 34+ & Structure  
03.12.14 Update SS & Final Release

Supersedes #: 34-8710-8114-9

Structure #: SS-13124

Reference: 90610-4, 6 & 9

Requesters: Scott Keppers

DEIE #: 24" x 14"

SSGS: 3818418 FILE: 34871429220.indd

PDF Scaled to 60%

Notes to Printer:  
Special Instructions:

## 3M™ Tegaderm™ Foam Adhesive Pansement Hydrocolloïdale Adhésif Haute Performance

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