3M™ Tegaderm™ Hydrocolloid Dressing and 3M™ Tegaderm™ Hydrocolloid Thin Dressing - Oval

Description
3M™ Tegaderm™ Hydrocolloid Dressing and 3M™ Tegaderm™ Hydrocolloid Thin Dressing are sterile wound dressings, which consist of a hydrocolloid, hydrocolloid adhesive with an outer adhesive clear or clear adhesive cover film. The film is moisture vapor permeable, waterproof and impermeable to liquids, bacteria, and viruses*. The products are not designed, used or intended for use except as indicated.

*In vitro testing shows that the transparent film of dressings provides a viral barrier from viruses 27 nm in diameter or larger while the dressing remains intact without leakage.

Over the wound site, Tegaderm™ Hydrocolloid Dressing is moistened with wound fluid to create a soft, semi-transparent, absorbent mass. The dressings maintain a moist wound environment, which has been shown to enhance wound healing.

Indications for Use
Tegaderm™ Hydrocolloid Dressing is indicated for partial and full thickness dermal ulcers, leg ulcers, superficial wounds, abrasions, first- and second-degree burns, and donor sites.

Tegaderm™ Hydrocolloid Thin Dressing is indicated for partial thickness dermal ulcers, leg ulcers, superficial wounds, abrasions, first- and second-degree burns, and donor sites. It may also be used as a protective dressing at risk, undrained skin or skin beginning to show signs of damage from friction or shear.

Contraindications
None known.

Warnings
None known.

Precautions
No se conocen.

1. Treatment of any skin ulcer should be part of a well-defined plan for ulcer management and under the supervision of a health care professional.

2. When using Tegaderm™ Hydrocolloid Dressing, the wound initially appears larger in size and depth as water-soluble tissue is cleared away. This increase should be accompanied by an improved appearance of the wound. If the wound gets larger after the first few dressing changes, see a health care professional.

3. Observe the wound for signs of infection. If infection occurs, the signs of which may be fever, increased pain, redness, bleeding, swelling, or an unusual odor or discharge see a health care professional. Tegaderm™ Hydrocolloid Dressings may be used on infected wounds only under the care of a health care professional.

4. Herpes, irritation (redness, inflammation), maceration (softening of the skin), or hypergranulation (excessive tissue formation) may develop in some wounds under hydrocolloid dressings. Should these occur, consult a health care professional.

5. If the wound does not begin to show signs of healing or if any other unexpected symptoms occur, consult a health care professional.

Directions for Use
Before Using the Dressing:

1. Only excess hair for patient comfort.
2. Clean the skin and the wound thoroughly.
3. Allow the skin to be dry before applying the dressing.
4. If the peripheral skin is fragile or exposed to trauma, dressing is likely, apply 3M™ Cavilon™ No Sting Barrier Film to the peripheral skin.
5. Evaluate the wound and select the correct dressing size so that 2.5 cm (1 inch) of the dressing, including the adhesive border, extends beyond the wound edge.

Applying the Dressing

1. Remove the paper liner from the dressing by lifting and pulling one of the square and taba marked "1", exposing the adhesive surface (Figure 1). Minimize contact with the border or the adhesive (bottom) side of the dressing.
2. Center the dressing over the wound. Gently press the adhesive (bottom) side down, and press from the center outward. Avoid stretching the dressing or the skin (Figure 2).
3. Press down and smooth the film edges to ensure good adhesion (Figure 3).
4. Remove the top delivery film by lifting and pulling one of the center tabs marked "2" toward the edge of the dressing (Figure 4). Smooth down the dressing edges as you are removing the film. Remove the other side of the top film in the same way.
5. Gently tear off the square and taba marked "1" at the perforation in a downward direction and discard (Figure 5). Avoid lifting the film edge while removing the tabs. Secure the entire film edge by pressing firmly.

Note: The dressing should be changed if it is leaking, falling off, or has been on the wound for seven days.

Applying the Dressing to Heals or Elbows:

1. Cut the oval dressing from one tab and to the center (Figure 6).
2. Secure the uncut half of the dressing to the bottom of the heel or elbow (Figure 7).
3. Press one half of the cut side of the dressing to the upper heel or the upper elbow (Figure 8).
4. Secure the top delivery film and the part of the dressing on the upper heel or elbow (Figure 9).
5. Secure the remaining half of the cut side of the dressing by overlapping it on the second portion of the dressing (Figure 10). Remove the remaining top delivery film and the tabs.
6. Secure the cut edge by pressing firmly (Figure 11).

Removing the Dressing:

1. Carefully lift the film edges from the skin. There is difficulty lifting the dressing, apply tape to the edge and use the tape to lift it.
2. Continue lifting the film until all edges are free from the skin surface.
3. Remove the dressing slowly, lifting it from the heel or the elbow in the direction of hair growth. Note: It is not unusual for wounds to have an odor. This may be noticed when the dressing is removed or when leakage occurs. The odor should disappear after the wound is cleaned.

Storage and Shelf Life
For best results, store at room temperature. Avoid excessive heat and humidity. For shelf life, refer to the expiration date which is printed on each package.

How Supplied
Individually packaged for single use only. Do not use if package is damaged or open.

If you have any questions or comments, in the USA contact the 3M Health Care Customer Helpline at 1-800-228-3957.

For further information, please contact your local 3M representative or contact us at 3M.com and select your country.

Explanation of Symbols

- Do not use if package is damaged or open
- Use by date
- The product and package do not contain natural rubber latex
- Manufacturer
- Caution, see instructions for use
- Date of manufacture
- Do not reuse
- STERILIZED using irradiation

CE 2797
Made in USA with globally sourced materials by
3M Health Care
2510 Conway Ave.
St. Paul, MN 55144
1-800-228-3957
3M.com/Tegaderm

3M Health Care
2510 Conway Ave.
St. Paul, MN 55144
1-800-228-3957
3M.com/Tegaderm

3M, Cavilon and Tegaderm are trademarks of 3M. Used under license in Canada.
© 2017, 3M. All rights reserved.
3M, Cavilon and Tegaderm sont des marques commerciales de 3M, utilisées sous licence au Canada.
© 2017, 3M. Tous droits réservés.
Issue Date: 2017-02
34-0726-0773-3

Pansements hydrocolloïdes ovales 3M™ Tegaderm™ Hydrocolloid et 3M™ Tegaderm™ Hydrocolloid Thin

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Description

Der 3M™ Tegaderm™ Hydrocolloid Dressing and 3M™ Tegaderm™ Hydrocolloid Thin Dressing are sterile wound dressings, which consist of a hydroallergic, hydrocolloid adhesive with an outer clear adhesive cover film. The film is moisture vapor permeable, waterproof, and impermeable to liquids, bacteria, and viruses*. The products are not designed, sold or intended for use except as indicated.

"In vitro testing shows that the transparent film of dressings provides a viral barrier from viruses 27 nm in diameter or larger while the dressing remains intact without leakage.

Over the wound site, Tegaderm™ Hydrocolloid Dressings interact with wound fluid to create a soft, semi-transparent, absorbent mass. The dressings maintain a moist wound environment, which has been shown to enhance wound healing.

Indications for Use

Tegaderm™ Hydrocolloid Dressing is indicated for partial and full thickness dermal ulcers, leg ulcers, superficial wounds, abrasions, first- and second-degree burns, and donor sites.

Tegaderm™ Hydrocolloid Thin Dressing is indicated for partial thickness dermal ulcers, leg ulcers, superficial wounds, abrasions, first- and second-degree burns, and donor sites. It may also be used as a protective dressing on at-risk, undamaged skin or on skin beginning to show signs of damage from friction or shear.

Contraindications

None known.

Warnings

None known.

Precautions

- Treatment of any skin ulcer should be part of a well-defined plan for ulcer management and under the supervision of a health care professional.
- When using Tegaderm™ Hydrocolloid Dressings, the wound initially appears larger in size and depth as wound fluid and tissue is cleared away. This increase should be accompanied by an improved appearance of the wound. If the wound gets larger after the first few dressing changes, see a health care professional.
- Observe the wound for signs of infection. If infection occurs, the signs of which may be fever, increased pain, redness, bleeding, swelling, or an unusual odor or discharge see a health care professional. Tegaderm™ Hydrocolloid Dressings may be used on infected wounds only under the care of a health care professional.
- Rarely, irritation (reddening, inflammation), maceration (whitening of the skin), or hypergranulation (excessive tissue formation) may develop in some wounds under hydrocolloid dressings. Should these occur, consult a health care professional.
- If the wound does not begin to show signs of healing or if any other unexpected symptoms occur, consult a health care professional.

Directions for Use

Before Using the Dressing:

- Clip excess hair for patient comfort.
- Clean the skin and the wound thoroughly.
- Allow the skin to dry before applying the dressing.
- If the periwound skin is fragile or exposed to drainage is likely, apply 3M™ Cavilon™ No Sting Barrier Film to the periwound skin.
- Evaluate the wound and select the correct dressing size so that 2.5 cm (1 inch) of the dressing, including the adhesive border, extends beyond the wound edge.

Applying the Dressing:

- Remove the paper liner from the dressing by lifting and pulling one of the square end tabs marked "1", exposing the adhesive surface (Figure 1). Minimize contact with the border or the adhesive (bottom) side of the dressing.
- Center the dressing over the wound. Gently press the adhesive (bottom) side down, and press from the center outward. Avoid stretching the dressing or the skin (Figure 2).
- Press down and smooth the film edges to ensure good adhesion (Figure 3).
- Remove the top delivery film by lifting and pulling one of the center tabs marked "2" toward the top edge of the dressing (Figure 4). Smooth down the dressing edges as you are removing the film. Remove the other side of the top film in the same way.
- Gently tear off the square end tabs marked "1" at the perforations in a downward direction and discard (Figure 5). Avoid lifting the film edge while removing the tabs. Secure the entire film by pressing firmly.

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Applying the Dressing to Heels or Elbows:

- Slit the oval dressing from one tab end to the center (Figure 6).
- Secure the uncult half of the dressing to the bottom of the heel or elbow (Figure 7).
- Secure one half of the cut side of the dressing to the upper heel or the upper elbow (Figure 8).
- Remove the top delivery film and tab from that part of the dressing on the upper heel or elbow (Figure 9).
- Secure the remaining half of the cut side of the dressing by overlapping it on the secured portion of the dressing (Figure 10). Remove the remaining top delivery film and the tabs.
- Secure the cut edges by pressing firmly (Figure 11).

Removing the Dressing:

- Carefully lift the film edges from the skin. If there is difficulty lifting the dressing, apply tape to the edge and use the tape to lift.
- Continue lifting the film until all edges are free from the skin surface.
- Remove the dressing slowly, folding it over itself. Pull carefully in the direction of hair growth.

Note: It is not unusual for wounds to have an odor. This may be noticed when the dressing is removed or when leakage occurs. The odor should disappear after the wound is cleaned.

Storage and Shelf Life

For best results, store at room temperature. Avoid excessive heat and humidity. For shelf life, refer to the expiration date which is printed on each package.

How Supplied

Individually packaged for single use only. Do not use if package is damaged or open.

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- | | |
|---|---|
|  |  |
|  |  |
|  |  |
|  |  |
| |   |

CE 2797

Made in USA with globally sourced materials by

3M Health Care
2510 Conway Ave.
St. Paul, MN 55144
1-800-228-3957
3M.com/Tegaderm

3M Canada
P.O. Box / C.P. 5757
London, Ontario N6A 4T1
1-800-364-3577

3M, Cavilon and Tegaderm are trademarks of 3M. Used under license in Canada.

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